



Yes, you can.

**SERVICE HISTORY RECORD
INVACARE OXYGEN CONCENTRATOR**

MODEL NO. IRC _____

SERIAL NO. _____



PREVENTIVE MAINTENANCE RECORD

INVACARE OXYGEN CONCENTRATOR

Yes, you can:

MODEL NO. IRC _____

SERIAL NO. _____

ON EACH INSPECTION

Record Date of Service																				
Record Elapsed Hours On Hour Meter																				
Clean Cabinet Filter(s)																				
Check Prescribed L/min. Flow Rate																				

EVERY 180 DAYS FOR IRC5LX (EVERY 90 DAYS FOR IRC10LX)

Check Oxygen Concentration (UNITS WITHOUT Senso₂ ®)																				
--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

ANNUALLY, DURING PREVENTIVE MAINTENANCE SCHEDULE, OR BETWEEN PATIENTS

Clean/Replace Cabinet Filter(s)																				
Check Outlet HEPA Filter *																				
Check Compressor Inlet Filter *																				
Check Oxygen Concentration (Senso₂ MODELS)																				
Check Power Loss Alarm																				

AS REQUIRED

Clean Heat Exchanger																				
Replace Exhaust Muffler																				
Rebuild Top End of Compressor																				

EVERY 20,000 HOURS

Replace In-line Filter - (ALL MODELS EXCEPT 3LX,5LX,5LXO2 5LXO2T, 6LXO2, 6LXO2T)																				
--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--	--

*Refer to Preventive Maintenance section of service manual, p/n 1118396