



# NAVIGATING THE SOLO<sub>2</sub><sup>™</sup> LCD SCREEN

## QUICK REFERENCE GUIDE


*NOTE: This guide is meant to be used only as a brief, general reference. For complete operating instructions, always refer to the Operator's Manual.*

### Turning Concentrator On/Off

*NOTE: For this procedure, refer to FIGURE 1.1.*

1. To turn the SOLO<sub>2</sub> concentrator on, press and hold the On/Off button  until the three warning lights display and the unit beeps.
2. To turn the SOLO<sub>2</sub> concentrator off, press and hold the On/Off button .


*NOTE: GREEN light will flash and you will hear the machine stop running.*

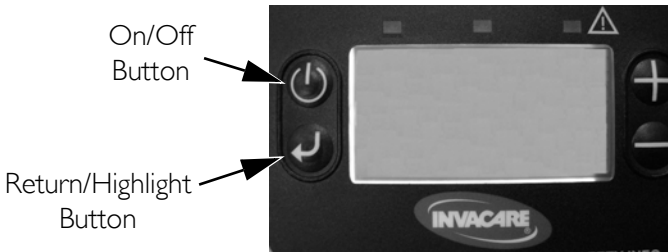

**FIGURE 1.1** Turning Concentrator On/Off

### Menu Screens Navigation

*NOTE: For this procedure, refer to FIGURE 1.1.*

1. After turning on unit, press and hold the Return/Highlight Button  to scroll through the three SOLO<sub>2</sub> concentrator menu screens.

*NOTE: Screens will continue scrolling until the Return/Highlight Button is released.*

### Normal Operating/Standby Screen

The Normal Operating/Standby screen shows either battery strength gauge, fan operating, external power on and battery charging icons, when applicable, or the current/last mode of operation and output flow level selected.

---

## LCD Adjust Screen







The LCD Adjust screen allows user to change the brightness and contrast of the screen (FIGURE 1.2).



**FIGURE 1.2** LCD Adjust Screen

## LCD Adjust Screen Navigation


*NOTE: For this procedure, refer to FIGURE 1.2.*

1. Perform one of the following, while the LCD Adjust screen is displayed:
  - A. Tap the Return/Highlight Button  to move the black highlight bar from left to right, or right to left.
  - B. Tap the Up/Increase Button  or the Down/Decrease Button  while the black highlight bar is on the "LEFT SIDE" of the screen to move the highlight bar up or down.
  - C. Tap the Up/Increase Button  or Down/Decrease Button  while the highlight bar is on the "RIGHT SIDE" of the screen to change the value of the item highlighted.
  - D. Hold down the Return/Highlight Button  to continue scrolling through the three menu screens.

*NOTE: If no buttons are pressed for approximately 10 seconds, the screen will change back to the Standby/Normal Operating screen. Any changes made will NOT be saved.*

---

2. To save changes made on the LCD Adjust screen:

- A. Press and hold the Return/Highlight Button  until a different screen appears.

## Mode & Flow Adjust Screen


The Mode & Flow Adjust screen allows user to change the current mode of operation and output flow (FIGURE 1.3).




**FIGURE 1.3** LCD Adjust Screen


## Changing Mode


*NOTE: For this procedure, refer to FIGURE 1.3.*

1. Press and hold the Return/Highlight Button  until the Mode & Flow Adjust screen appears.


*NOTE: Verify that "MODE" is highlighted by the black highlight bar before proceeding.*

2. Tap the Return/Highlight Button  to move the highlight bar to the right. The current mode ("CONTINUOUS" or "PULSE") should now be highlighted.

- A. If "CONTINUOUS" is highlighted, press the Down/Decrease Button  to change the mode to "PULSE".

- B. If "PULSE" is highlighted, press the Up/Increase Button  to change the mode to "CONTINUOUS".







3. To save changes made on the Mode & Flow Adjust screen:

- A. Press and hold the Return/Highlight Button  until a different screen appears.

---

## Adjusting Flow

*NOTE: For this procedure, refer to FIGURE 1.3.*

1. Press and hold the Return/Highlight Button  until the Mode & Flow Adjust screen appears.
2. Tap the Down/Decrease Button  to move the highlight bar down. "FLOW" should now be highlighted.
3. Tap the Return/Highlight Button  to move the highlight bar to the right. The current flow rate should now be highlighted.
4. Tap Down/Decrease Button  to decrease flow, or tap the Up/Increase Button  to increase flow as needed.
5. To save changes made on the Mode & Flow Adjust screen:
  - A. Press and hold the Return/Highlight Button  until a different screen appears.

*\*NOTE: If no buttons are pressed for approximately 10 seconds, the screen will revert back to the Normal Operating/Standby screen, undoing any un-saved changes.*

*\*NOTE: The SOLO<sub>2</sub> concentrator will default to the last flow rate saved for each mode setting.*

*\*NOTE: The LCD Adjust screen follows the same principles described above.*



**Yes, you can:**

**Invacare Corporation**

[www.invacare.com](http://www.invacare.com)

**USA**

One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900  
Technical Services  
800-832-4707

All rights reserved. Trademarks are identified by <sup>™</sup> and <sup>®</sup>. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

© 2009 Invacare Corporation

Part No | 163220

Rev A - 8/09

