

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.

Safety Summary

To ensure the safe and proper use of the XPO₂ supplemental battery, these instructions **MUST** be followed:

⚠ WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Use only with Invacare XPO₂ portable oxygen concentrator.

Connecting the Supplemental Battery to the XPO₂

NOTE: For this procedure, refer to FIGURE 1.

1. Connect the supplemental battery pack to the XPO₂ as shown in FIGURE 1. Ensure the supplemental battery is fully charged prior to initial use.

NOTE: . This supplemental battery pack will provide 2.5 hours of additional run time at setting 2. This supplemental battery pack has its own battery status button and battery gauge to identify the level of battery charge.

NOTE: The supplemental battery cannot be used to charge the XPO₂'s internal battery.



FIGURE 1 Connecting the Supplemental Battery to the XPO₂

For more information regarding Invacare products, parts, and services, please visit www.invacare.com

Charging the Supplemental Battery

NOTE: For this procedure, refer to FIGURE 2.

1. Connect the AC or DC power adapter to the supplemental battery pack.
2. Connect the other end of the AC or DC power adapter into the appropriate power source.

NOTE: If completely discharged, the supplemental battery will fully charge in approximately 4 hours.



FIGURE 2 Charging the Supplemental Battery

Charging the Internal and Supplemental Battery Pack Simultaneously

NOTE: For this procedure, refer to FIGURE 3.

1. To charge the supplemental battery pack and internal battery together, perform the following:
 - A. Connect the AC or DC power adapter to the supplemental battery pack as shown.
 - B. Connect the supplemental battery pack to the XPO₂ as shown.
 - C. Plug the other end of the adapter into the power source.

NOTE: The internal battery charge status will be indicated by the XPO₂'s battery gauge and the supplemental battery charge will be indicated by the illuminated lights located on the supplemental battery pack assembly.



FIGURE 3 Charging the Internal and Supplemental Battery Pack Simultaneously

NOTE: You may operate your XPO₂ Portable Concentrator while the internal and/or the supplemental battery is being charged. Use of the XPO₂ during charging will lengthen the amount of time required to reach full charge status of the batteries.

TYPICAL PRODUCT PARAMETERS

Dimensions	6.8 in high x 2.5 in wide x 1.8 in deep (17.1 cm high x 6.4 cm wide x 4.4 cm deep) Without molded cord
Weight	1.3 lbs (0.68 kg)
Battery Recharge:	4 Hours (Recharge time increases if battery is charging while unit is running. Times are approximate)
Battery Duration: (When used with discharged internal battery, times are approximate)	Setting 1 = 3.5 hrs
	Setting 2 = 2.5 hrs
	Setting 3 = 2.0 hrs
	Setting 4 = 1.5 hrs
	Setting 5 = 1.0 hrs
Battery Duration: (When used with fully charged internal battery, times are approximate)	Setting 1 = 7.0 hrs
	Setting 2 = 5.0 hrs
	Setting 3 = 4.0 hrs
	Setting 4 = 3.0 hrs
	Setting 5 = 2.0 hrs
Humidity:	Operating humidity: 15% to 60% non condensing
	Storage humidity: up to 95% non condensing
Temperature Range:	Operating Temperature: 41°F to 95°F (5°C to 35°C)
	Storage Temperature: -2°F to 140°F (-20°C to 60°C)
Extended Temperature Range (Used with fully charged internal battery)	Operating Temperature: 95°F to 104°F (35°C to 40°C) Continuous use - all settings
Extended Temperature Range (When used with discharged internal battery)	Operating Temperature: 95°F to 104°F (35°C to 40°C) Continuous use - settings 1,2,3 30 minutes (max) - setting 4 20 minutes (max) - setting 5

LIMITED WARRANTY

NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare Corporation or a dealer. This warranty is not extended to any other person or entity and it is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. This warranty gives you specific legal rights and you may also have other legal rights which may vary from state to state.

Invacare Corporation warrants the supplemental battery when purchased new and unused to be free from defects in materials and workmanship for a period of one year from date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty.

DO NOT OPEN OR ATTEMPT TO SERVICE UNLESS SERVICE IS PROVIDED BY AN INVACARE-CERTIFIED TECHNICIAN; otherwise, this will void any and all warranties. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty only applies to the

labor for repairs performed by the Invacare Service Department. It does not apply to the labor performed by the purchaser or user. This warranty does not include normal wear and tear or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Routine maintenance items, such as filters, are excluded from this warranty.

For warranty service, please contact Invacare's service department at the toll free number on the back page during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare Corporation will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return the entire unit or remove, at purchaser's cost, the defective component part(s) identified, pack the component part(s) in a manner to avoid shipping damage and to ship the component part(s) to either Invacare Corporation's plant or service center as specified by Invacare Corporation in advance. Defective component part(s) **MUST** be returned for warranty inspection using the serial number as identification within thirty days of return authorization date. **DO NOT** return products to our factory without prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, SOOT OR SMOKE-FILLED ENVIRONMENTS, OR OTHER THAN NORMAL APPLICATION, USE OR SERVICE, OR TO PRODUCTS MODIFIED WITHOUT INVACARE CORPORATION'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE CORPORATION OR TO PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE CORPORATION'S CONTROL.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT APPLY TO YOU.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.



Yes, you can.

Invacare Corporation

www.invacare.com

USA

One Invacare Way
Elyria, Ohio USA
44036-2125
800-333-6900

Canada

570 Matheson Blvd E Unit 8
Mississauga Ontario
L4Z 4G4 Canada
800-668-5324

All rights reserved. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2007 Invacare Corporation

Part No 1151577

Rev A- 10/07